



VT009 Handlebar Installation Guide (Basic Version)

This handlebar installation is a bit too tedious.

I apologize for not having a more user-friendly design at this time.


Hope this installation guide is clear and helpful.

Video tutorial is available. Please scan the QR code or enter the URL below to view.

Thank you so much!

Jay Tang / Engineer
Cell: 651-492-5007 (call or text)
jay@vibrationtherapeutic.com

Video Tutorial

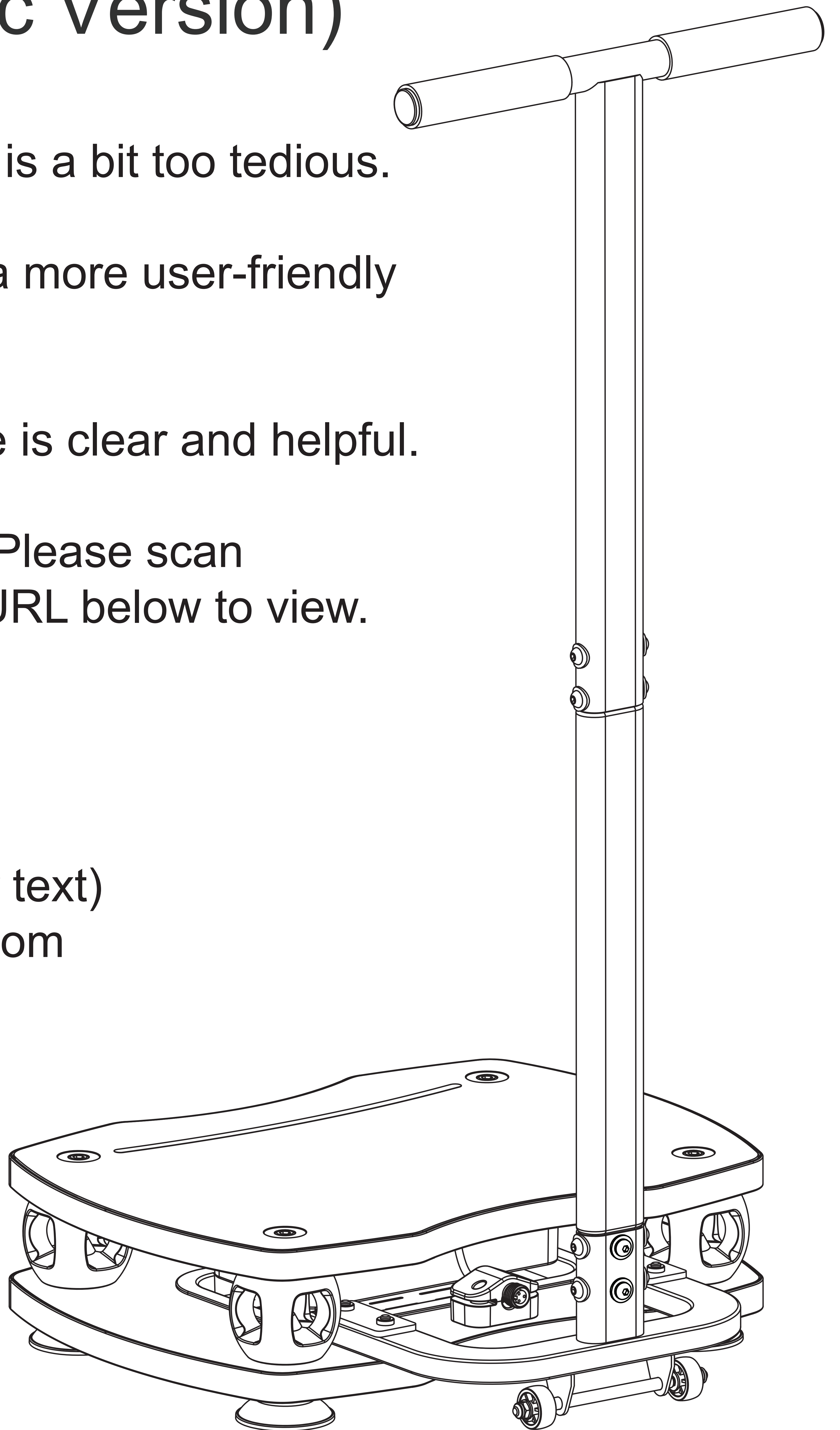


www.vibrationtherapeutic.com/vt009-handlebar/

1

Explosive View


Video Tutorial

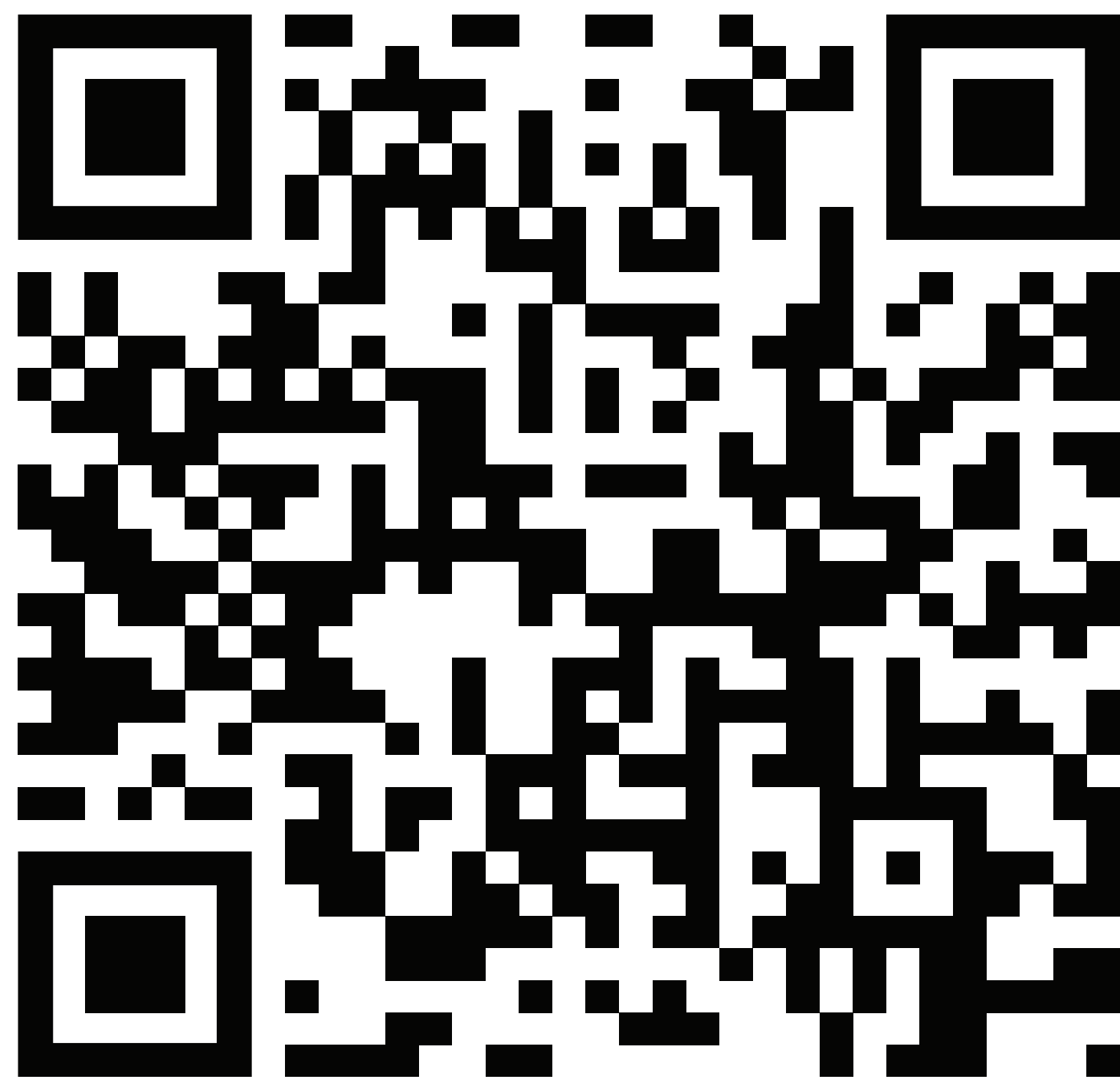

T-Bar

Mid-Tube








Base

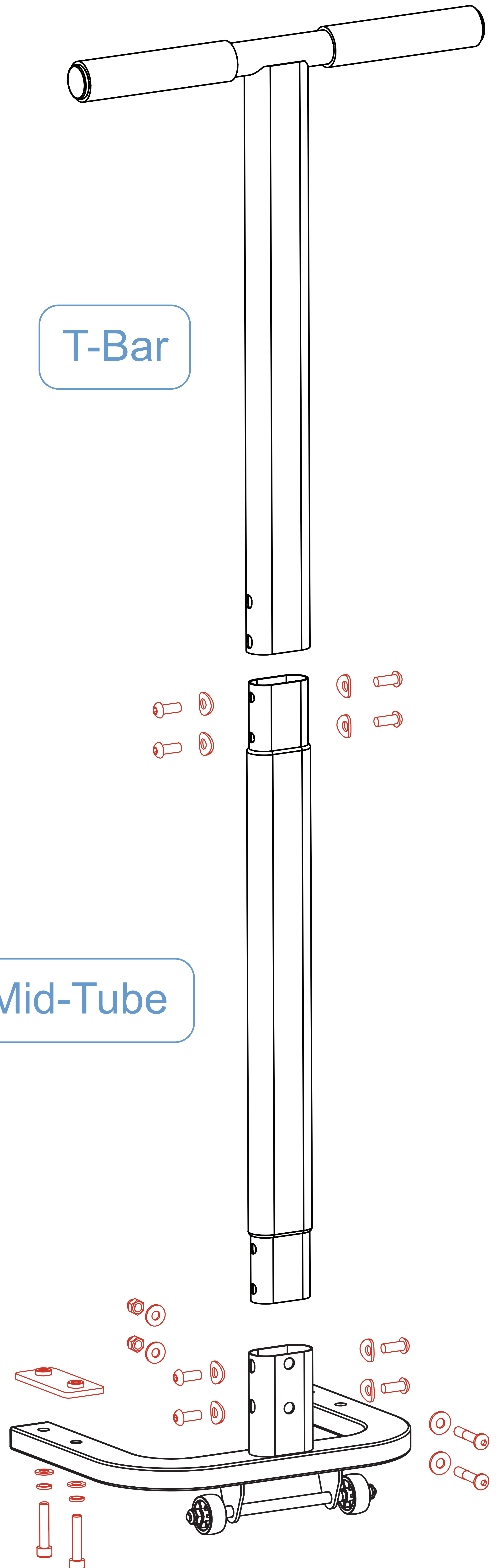




Video Tutorial: www.vibrationtherapeutic.com/vt009-handlebar/

Hardware & Tool Kits

VT009 Handlebar Hardware Kit (1)

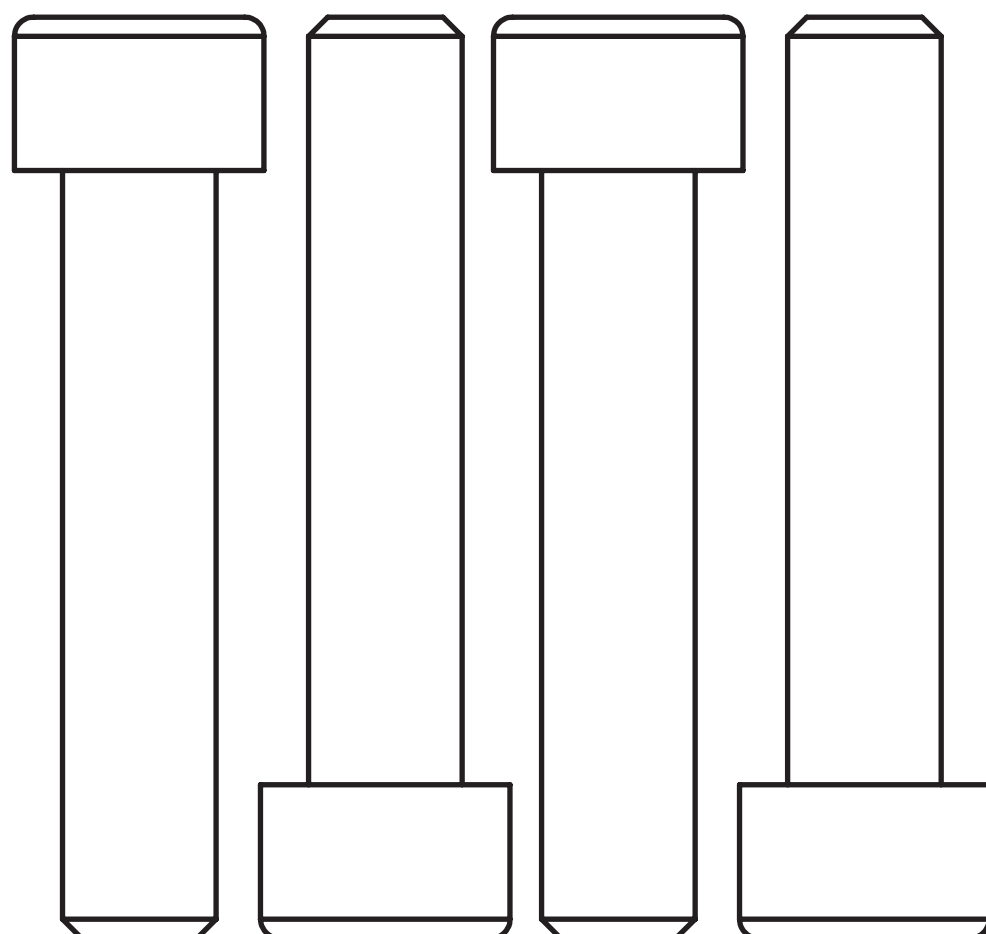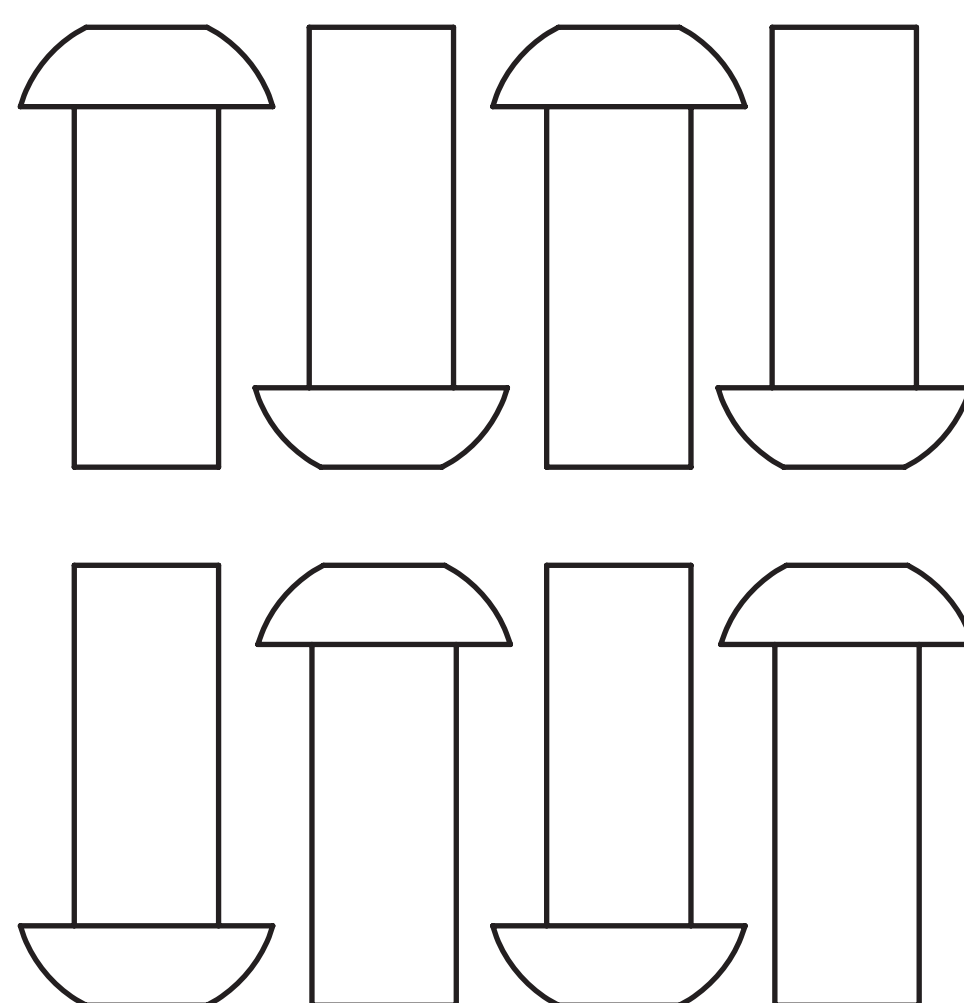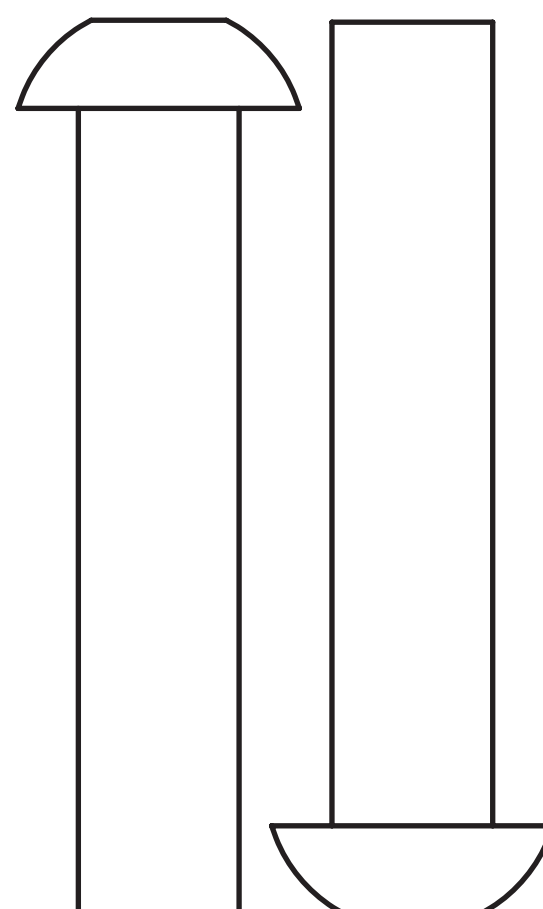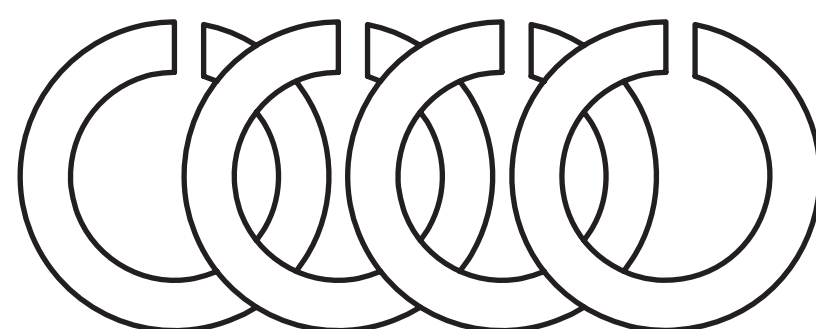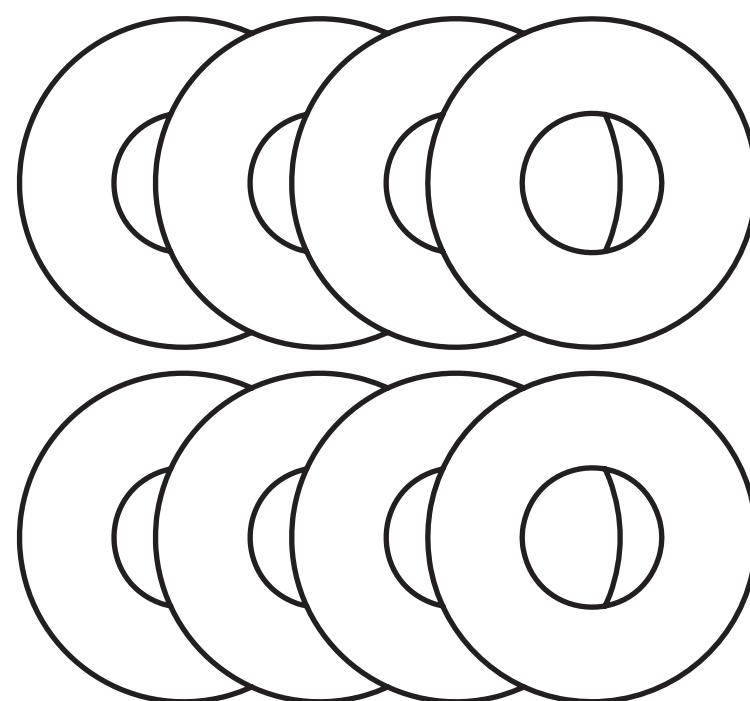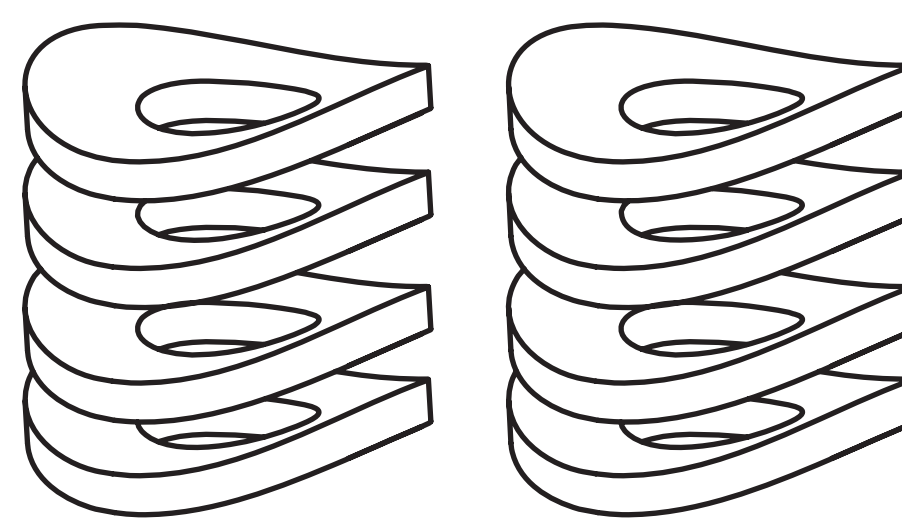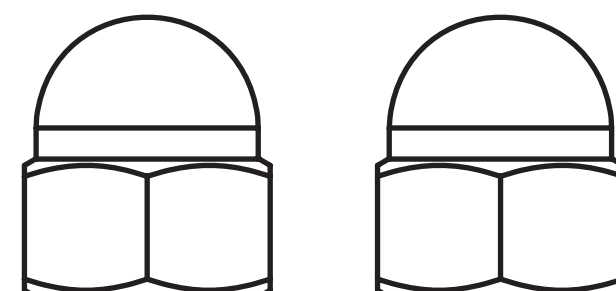
A Bolts (40mm)  4 pcs for Base	D Screw (20mm)  8 pcs for Mid-Tube	F Bolt (40mm)  2 pcs for Mid-Tube	
B Lock Washers  4 pcs for Base	C Flat Washers  8 pcs for Base	E Curved Washers  8 pcs for Mid-Tube	G Cap Nuts  2 pcs for Base

VT009 Handlebar Hardware Kit (2)

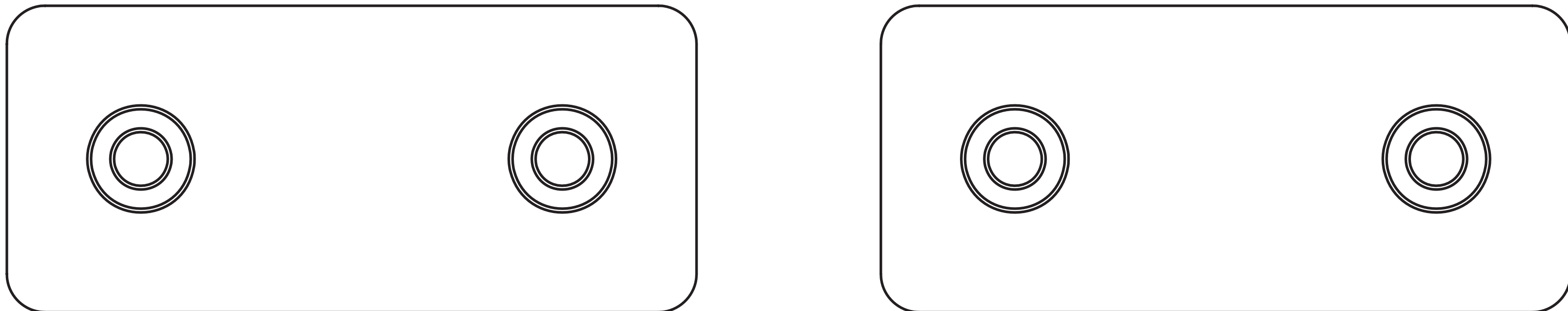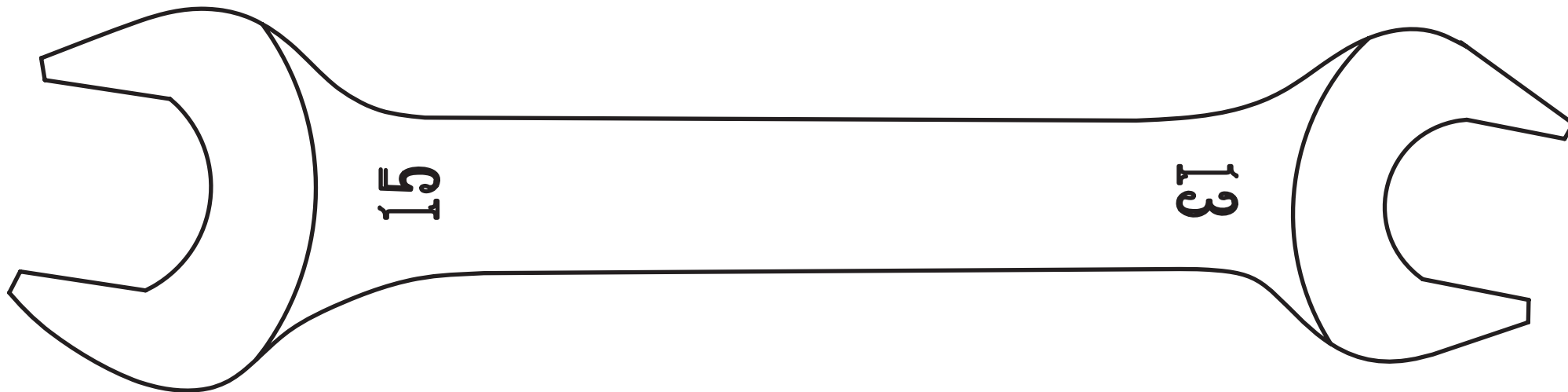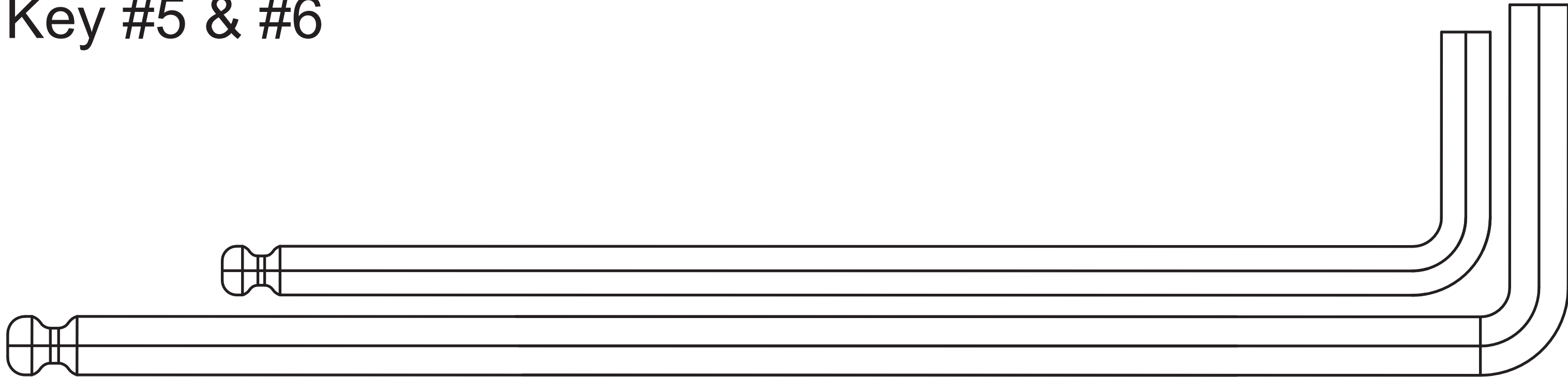H Bracket with Nuts x 2 for Base 
I Wrench #13/15 for cap nuts (E) 
J Allen Key #5 & #6 


3

Get Prepared

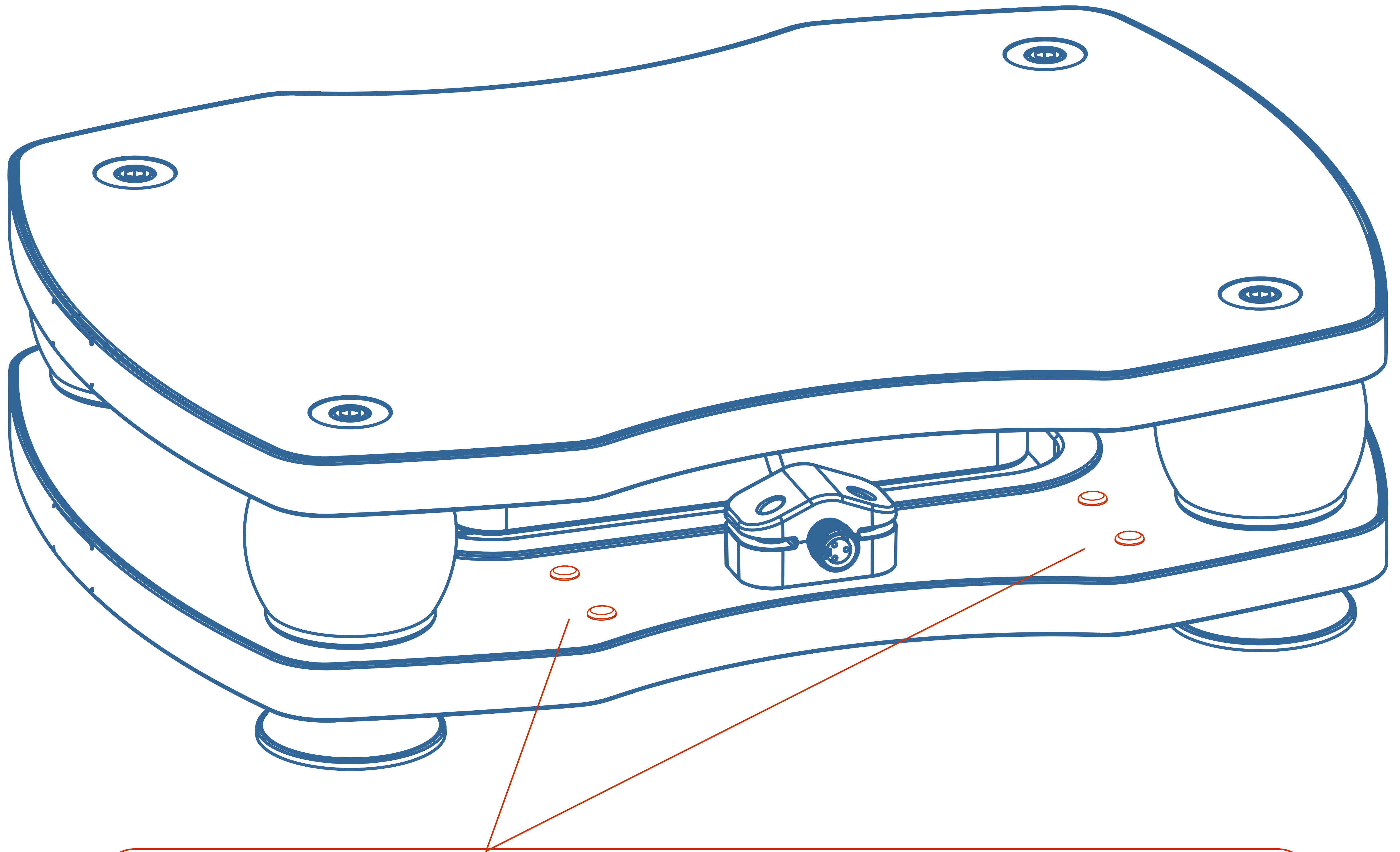


Remove the plastic caps before installation.

Where the Base is installed?

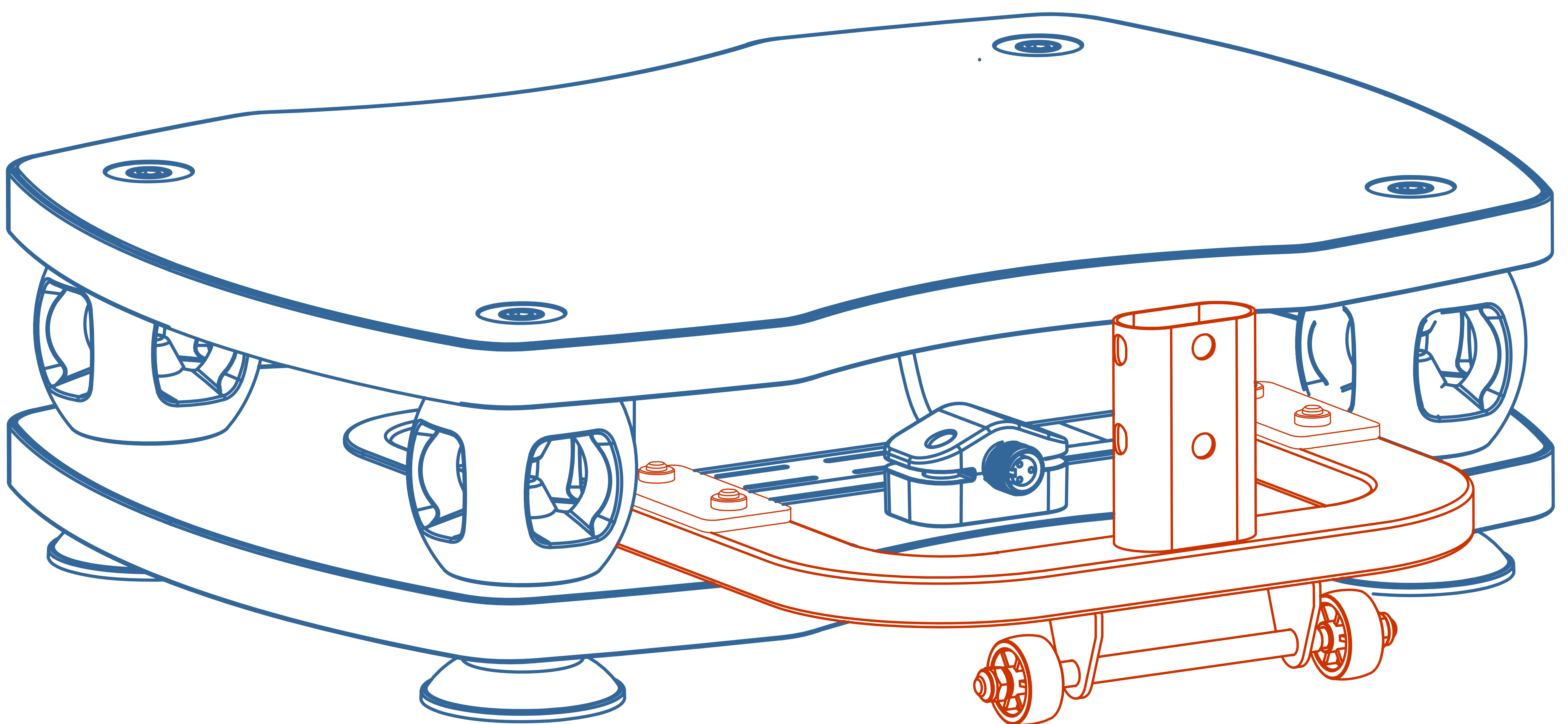


4

Install the Base

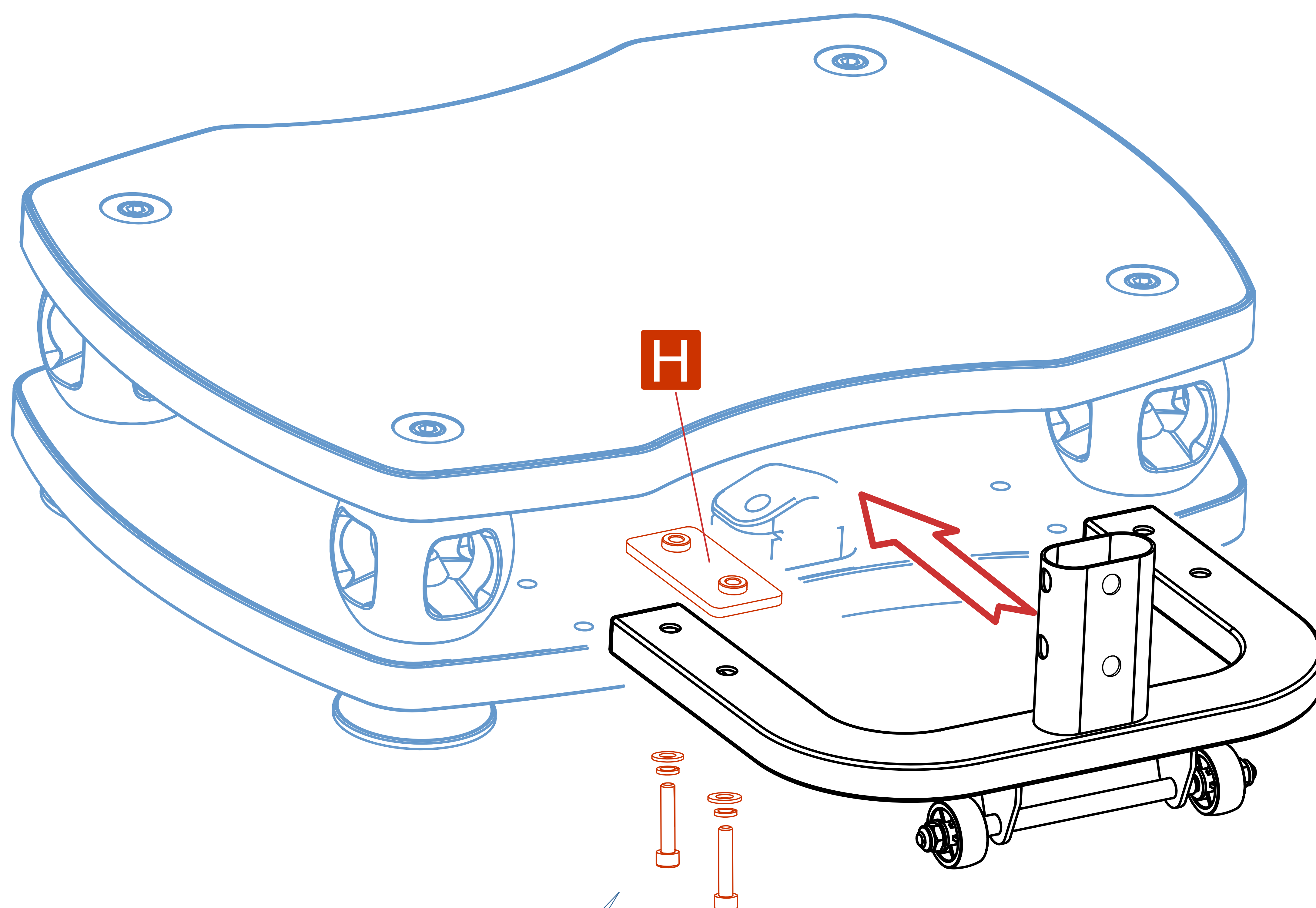


Flip up the VT009 for easy operation

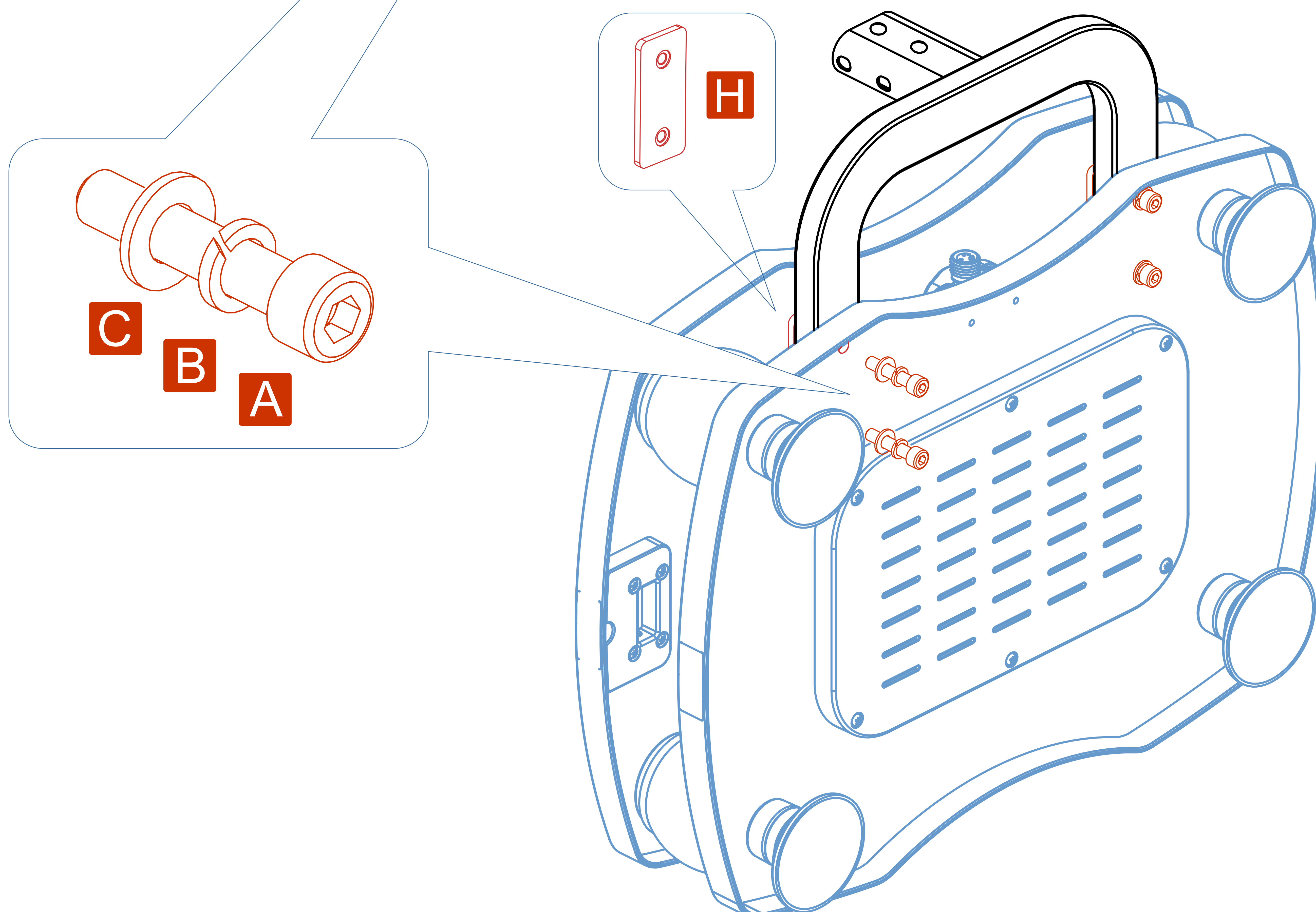


5

Insert the Mid-Tube

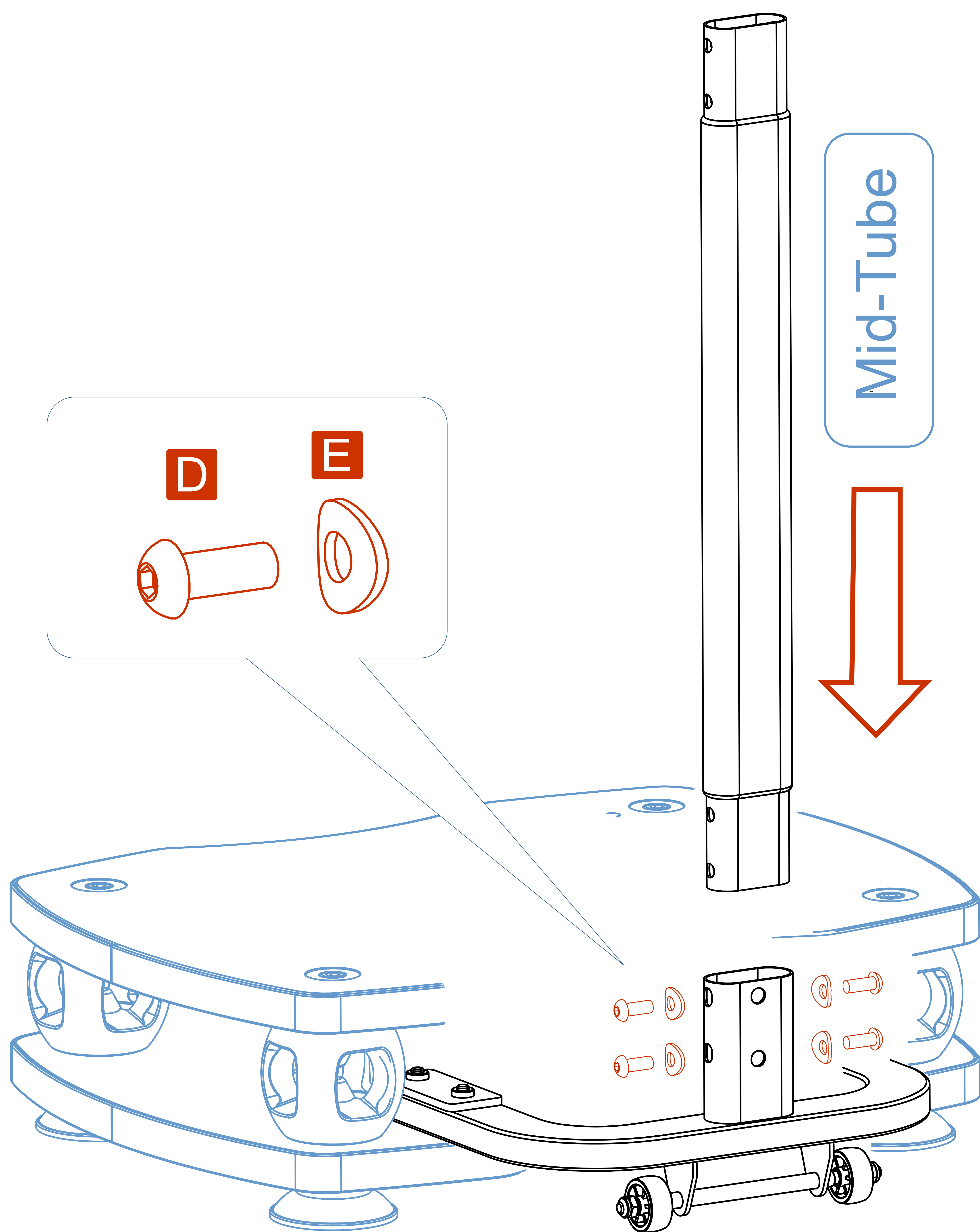


6

Add additional bolts and nuts

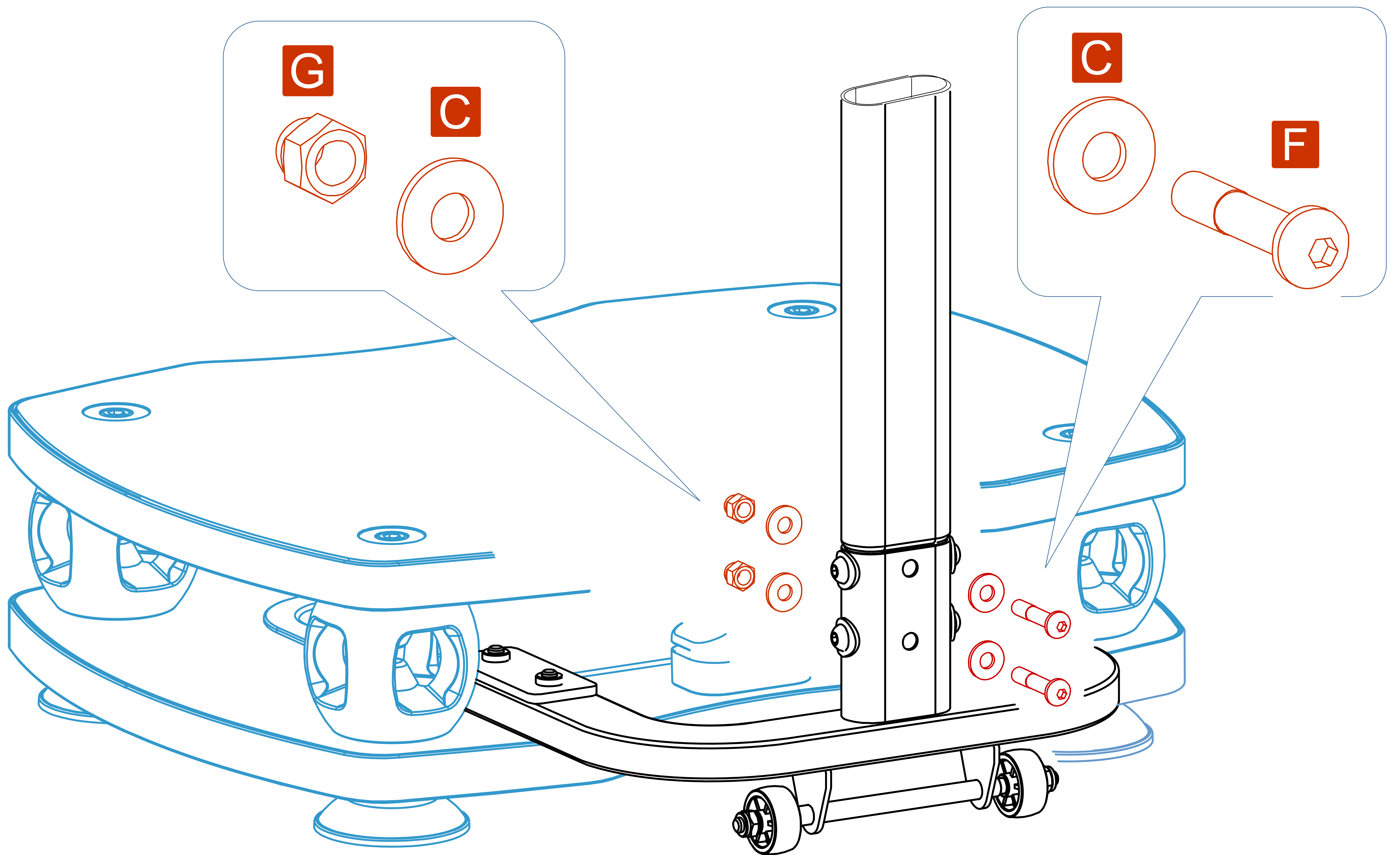


Side View

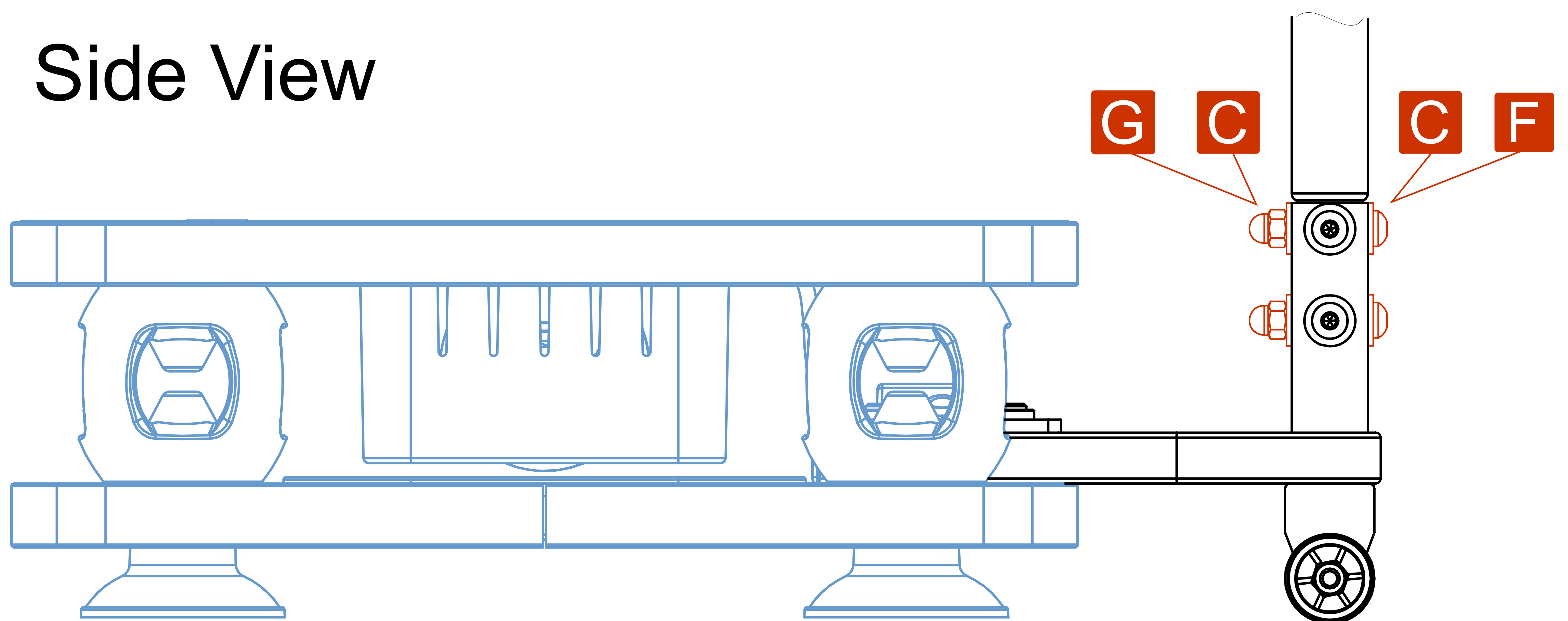

7

Insert the T-Bar

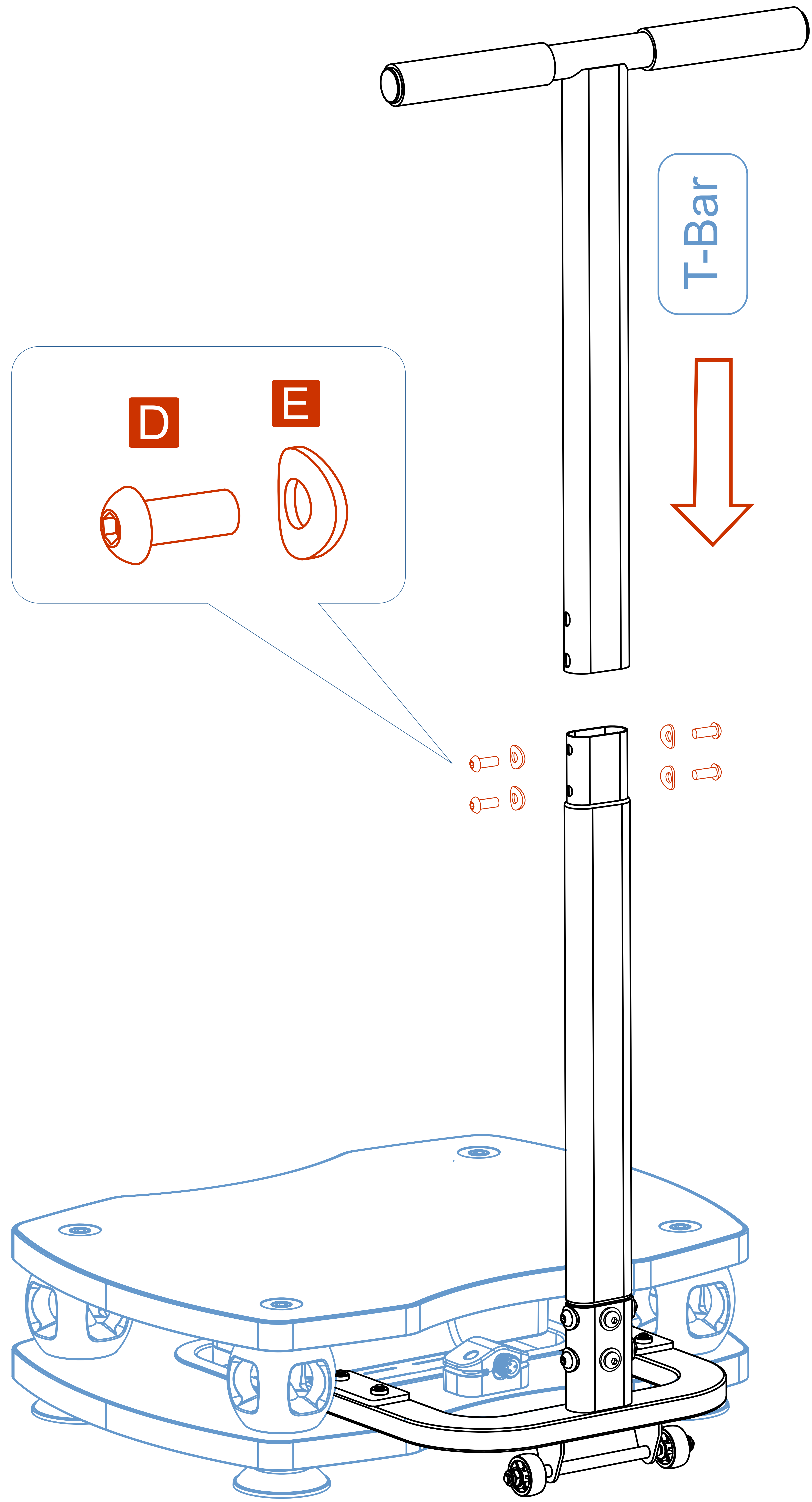

8

You did it!

Make sure the screws are tight!

Hope you enjoy using the VT009.

Visit www.vibrationtherapeutic.com
for product applications.

Thank you!

